



## “mediliev touch” and “mitouch” Device Cleaning Instructions.

Maintaining “mediliev touch” and “mitouch” units and accessories ensures maximum device longevity and safe, hygienic home therapy sessions. Proper care focuses on three distinct components: the device itself, the lead wires, and the reusable electrode pads. Each Kor accessory also has its own proper care component.

Instructions for use (IFUs) and treatment applications are included in the device boxes, on the backs of accessory boxes, and in the Veteran Protocol Guide. Device electrical information is included in the device box (paper insert) and on the back of the accessory boxes.

### 1. Cleaning the “mediliev touch and mitouch” Device and Lead Wires

- **Device Control Unit:** Gently wipe the digital casing with a soft sanitizer wipe or disinfectant wipe. Never immerse the unit in water or other liquids, and avoid harsh chemical cleaners or abrasive agents.
- **Lead Wires:** Wipe the cords with a sanitizing or disinfecting wipe. If they become tangled or sticky, coat them lightly with talcum powder to prolong their lifespan.

## 2. Cleaning and Caring for Electrode Pads (Snap Electrodes are for single-patient use only)

- **Restoring Stickiness:** If the pads lose adhesion due to dust or natural body oils, gently rub a few drops of water on the gel surface with your fingertip for 10–15 seconds to hydrate them. Let them air dry completely.
- **Removal:** Turn the device off completely before removing the pads from your skin. Always peel them from the edge rather than pulling on the attached wire.
- **Storage:** Always place the pads back onto their clear plastic film (gel side down) and seal them in their provided polybag. Keep them stored in a cool, dry place away from heat or direct sunlight.

## 3. Pre-Treatment Skin Care

- **Clean and Dry:** Before applying the electrodes, clean the treatment area with mild soap and water or an alcohol-based disinfectant wipe. Let the treatment area dry completely.
- **Oils and Lotions:** Ensure your skin is free from natural oils, lotions, or creams, as these degrade the adhesive gel on the pads.

## 4. “KorBand” Accessory:

- **Skin Preparation:** Clean the skin with an alcohol-based disinfectant wipe or mild soap and water prior to use.
- **Accessory Preparation:** Wipe the surface of the KorBand pad (carbon-rubber side) and the lead wires with a

disinfectant or sanitizer wipe. Let the disinfectant or sanitizer on the KorBand pad (carbon-rubber side) dry prior to use.

## 5. “**KorShoes**” Accessory:

- **Skin Preparation:** Clean the skin with an alcohol-based disinfectant wipe or mild soap and water prior to use.
- **Accessory Preparation:** Wipe the surface of shoes and lead wires with a disinfectant wipe. Let the disinfectant on KorShoes dry prior to use.

## 6. “**KorGlov and KorSocks**” Accessories:

- **Skin Preparation:** Clean the skin with an alcohol-based disinfectant wipe or mild soap and water prior to use.
- **Cleaning textile garment accessories:**
  - Hand wash accessories with mild soap and air dry only.
  - **DO NOT** place in washing machine or machine dryer.
  - **DO NOT** use disinfectant on textile garment accessories.
- **Intended for single-patient use.**